


Smartz™ Quick Start

1. Collect the Smartz™ pod and Smartz™ pad you wish to use.

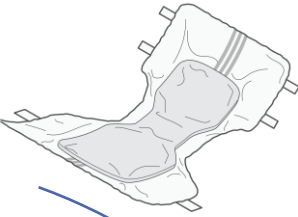


2. Create your User account following the prompts of the Smartz™ app or log in with your User details.




3. Add the Patient's details and Manufacturer code.



4. Check the Smartz™ pod number shown on the inside of the lid e.g. SN-12-3456. Assign the pod and pads.

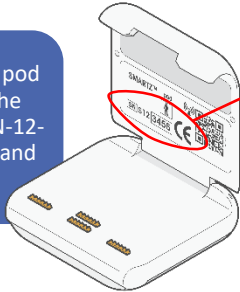

5. Connect pod to pad and watch for the green light flash on pod. Check the green light in the app.

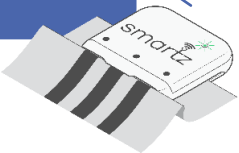

6. View notifications of Smartz™ pad status and other action items on the app or Smartz™ dashboard.



7. Review messages via the Smartz™ Notification app.

